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For more details of our variable seat and back systems and accessory options see on page ...

	UNITOMIC® ANATOMIC SEAT & BACK-SYSTEM	36/37
	MOFLEX FLEXIBLE-POSITIONING	112/113
	TRIKE OUTDOOR-EXTENSION	126 – 129
	FIDIBUS ANATOMIC SEAT & BACK-SYSTEM	130 – 133
	MATRIX ANATOMIC SEAT & BACK-SYSTEM	134 – 137
	TABLE THERAPY TRAY	138/139
	CRASH TESTED <small>ISO 7176-19</small>	crash-getestet nach ISO 7176-19

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General informations

Please note, that people with special needs, reading difficulties and impaired vision must ask their parents, careers or technical advisors to read or explain this technical documentation, so that safe usage of our products can be guaranteed.

We would like to point out, that only spare parts and accessories from our company may be used and installed so as not to endanger the safety of the users. In individual cases, the combination of our products with those of other manufacturers may require clarification with your specialist retailer or with Berollka.

Settings and adjustments to the product or accessories may only be made by people who have been given the necessary instructions by a medical product advisor. Please ensure that none of the user's extremities are in the respective area when making adjustments to avoid the risk of injury.

Note: We reserve the right to change design, color and material according to the state of the art and within the framework of customary product development. The illustrations of the products and colors are for guidance only. Also the products in the catalog partly show special equipment, that deviates from the standard equipment. Deviations from the original can be caused by different media (e.g. monitor display, print etc.).

All information is provided without warranty.

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Dear customers, interested people and partners,

Berollka bewegt – the guiding principle of our certified owner-managed family business in the second generation is:

Improve quality of life through innovation and individual solutions, so that a free, independent life for people with special needs is not a dream anymore.

For this we set new standards with heart, expertise and know-how in the development and production of wheelchairs, mobility bases and standers.

This catalog offers you an insight into the current range of our products and the opportunity to get comprehensive information about the different product variants.

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Flexibility and independence are important prerequisites for cognitive and emotional child development. In order for children with reduced mobility not to do without, Berollka produces wheelchairs that can be adjusted to individual needs. This ensures that those children are

more easily integrated in daily life such as school and leisure time. The wheelchairs are technically state of the art so that the children feel completely comfortable and thus can improve their individual skills more easily during their development.

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The smallest for the first exploring

The new children's wheelchair KAYOU has been specially adapted to the needs of infants. The infants should be enabled on more mobility and activity, because the development of the child between the 1st and 2nd year of life is therapeutically very important. This is the basis for future development.

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For explanations see first flip page of this catalog.

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*KAYOU anti-tip wheel for
narrow seat widths also as
a middle version*

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*in HighLine design with coloured
seat and back upholstery,
coloured footplate
and handrims*

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- *lightweight alu frame with abduction of 12 cm*
- *aluminium side guards with clothing guard made of plastic*
- *aluminium footrest plate and seatplate*
- *side guard with integrated brake*
- *huge range of accessories and add-ons*

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Details

Aluminium frame with large abduction of 12 cm. Easily stepless adjustable seat and back geometry and lower leg length.

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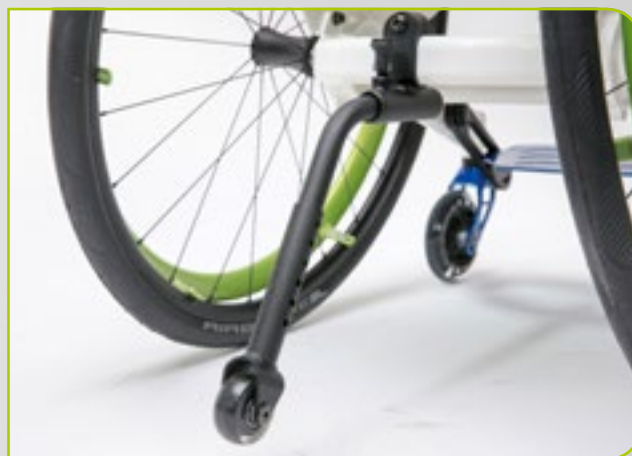
KAYOU Sitzsystem
KAYOU seating system

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The little racer with the sporty design

The new rigid-frame wheelchair FILOU is very agile and easy to drive due to its dynamics and low weight. The FILOU was designed with its rigid frame for children who do not avoid any adventure.

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*with frame shape H1 with knee
angle brake integrated
in the side guard*

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- *rigid wheelchair made of aluminium*
- *straight or abduction frame in different frame lengths and frame heights*
- *back angle adjustable*
- *huge range of accessories and add-ons*

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Details

This wheelchair is characterized by excellent functionality in all areas. It also guarantees ergonomic seating and extraordinary handling thanks to the optional ergonomic back unit UNITOMIC®.

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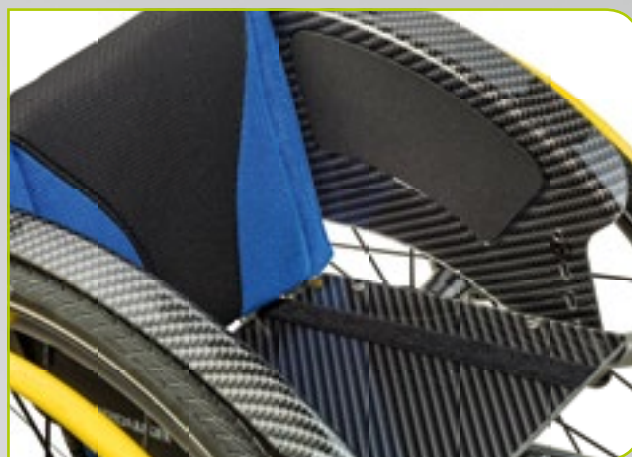
*More details see on page 36/37
and from page 124.*

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Convincing driving characteristics and optimal adjustability

The new child and youth wheelchair FINDUS can be used in a variety of ways. With the new growing rigid frame an extraordinary dynamic and mobility in all situations is easily possible.

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* depending on frame length
** depending on seat width

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back upholstery and seat cushion with SlimLine design

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*with FIDIBUS back system
in HighLine design with upholstery
of breathable fabric*

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- *rigid aluminium frame (growing along with the child) available in straight or abduction frame*
- *special frame MECKI or EXTREME*
- *optional back angle adjustable and foldable*
- *huge range of accessories and add-ons*

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Details

With its low weight and its agility, the FINDUS can be handled perfectly in everyday use. This wheelchair has been developed over many years, satisfying the needs of children, parents and therapists alike.

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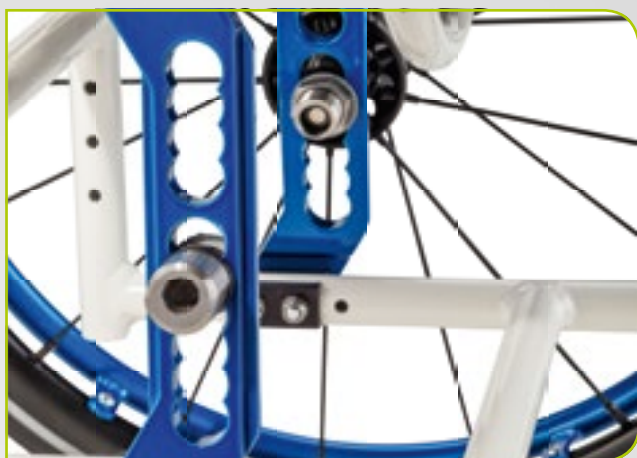
The sporty lightweight

The new child and youth wheelchair SLASH can be used in a variety of ways. With the new double tube frame and the stepless adjustable drive unit an extraordinary dynamic and flexibility in all situations is easily possible. Due to its low weight, the SLASH is very agile and can be driven very sporty.

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*in HighLine design with SlimLine upholstery,
coloured forks, footrest plate and
handrims*

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- *rigid aluminium frame, available in straight or abduction*
- *different footrest plate variations*
- *FIDIBUS seat and back system optionally available*
- *huge range of accessories and add-ons*

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Details

The innovative design combined with an ultra-light construction makes many things easier on trips and in the everyday life.

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The manoeuvrable light-weight for easy mobility

The trendy design and its easy handling make the LITTLE a real newcomer among childrens wheelchairs. The LITTLE offers, through its great freedom of movement, active mobility for children. With its low weight, the LITTLE adapts optimally to the child. The new frame gives the wheelchair a joyful ride and high stability.

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*in HighLine design
with ERGO AL clothing guard*

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*in HighLine design with SlimLine
upholstery coloured forks
and footrest plate*

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- *foldable wheelchair made of aluminium with integrated or removable footrests*
- *straight or abduction frame in different frame lengths*
- *huge range of accessories and add-ons*

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Details

The innovative design combined with an ultra-light construction makes many challenges in the everyday life easier.

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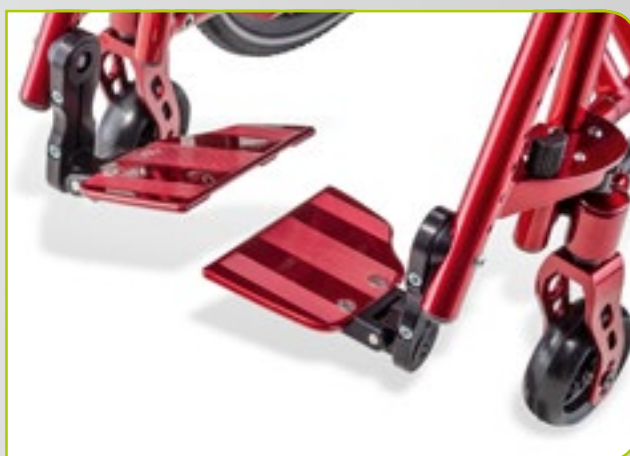
Abduktion LITTLE SLT
LITTLE SLT with abduction

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Abduktion LITTLE AR
LITTLE AR with abduction

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Playfully active, every day

Active mobility and maximum freedom of movement are particularly important for children. The JUNIOR2 wheelchairs allow a maximum of child-friendly mobility – in all situations of everyday life.

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*For explanations see first flip
page of this catalog.*

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- *foldable wheelchair made of aluminium with integrated or removable footrest*
- *straight or abduction frame in different frame lengths*
- *huge range of accessories and add-ons*

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Details

The solid construction ensures exact driving characteristics and easy handling. By replacing fewer components, an individual adaption to the growth of the child is possible. Through technology in perfection and a sporty design, the JUNIOR2 wheelchairs are the ideal companion and let not only children's heart beat faster.

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Fast sprints or laid-back

The YUKI enables active mobility and the greatest freedom of movement in combination with much more comfort. The modern design, with clever details, makes many things easier in everyday life.

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back with adjustable back upholstery and integrated push handles

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mechanically seat tilt with triggering on push handle and mechanical back angle adjustment

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- *rigid wheelchair made of aluminium available in two frame lengths*
- *seat frame angle adjustable*
- *back angle adjustable*
- *back foldable*

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Details

Due to the seat tilt and the angle-adjustable back, it is possible to switch from an active sitting position to a relaxed resting position at any time. With optional attachments, such as headrest, armrests and therapy table, the Yuki can be ideally adapted to the needs of the child.

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Sprints or relaxation

The TRAXX allows active mobility and great freedom of movement with a lot of comfort. The modern design, which is thought out in every detail, makes many things easier in everyday life.

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*with single push handles
height adjustable and back
upholstery adjustable*

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*with seat angle adjustment by gas spring and
mechanical back angle adjustment*

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- *rigid wheelchair made of aluminium available in two frame lengths*
- *seat frame angle adjustable (by gas spring)*
- *back angle adjustable*
- *back foldable*

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Details

With the tiltable seat, a comfortable rest position can be set. For both frame lengths, the seat depth can be changed steplessly. The back can be chosen in different back angles.

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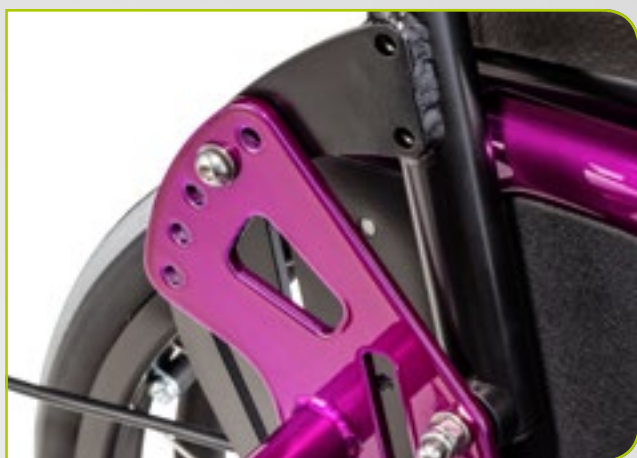
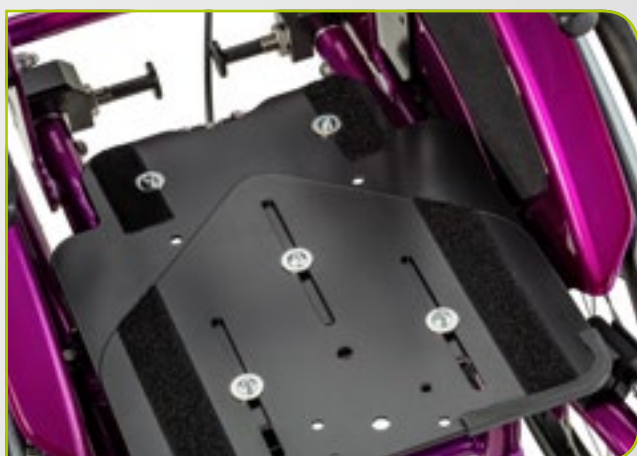
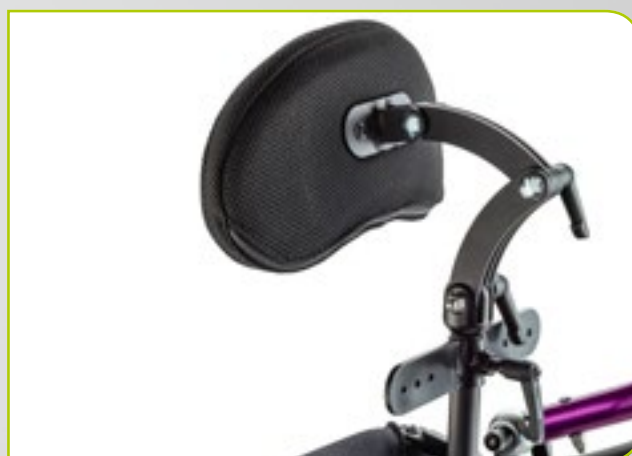
The anatomic adaptable back system

Who sits long, needs a healthy seating position. With a wrong position, back tensions are predestined and the musculature does not longer support the spinal column.

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upright position with a healthy relationship between tension and relaxing.

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"hunched back" causes tensions and back pain.

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The UNITOMIC® back system prevents poor postures. It gives a healthy relationship between tension and relaxing of the different muscle groups. In addition, the UNITOMIC® prevents the position of a hunched back.

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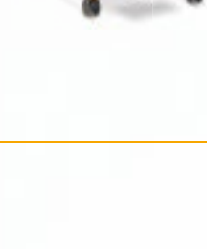
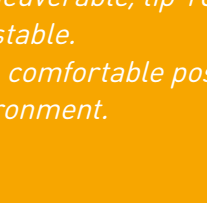
Berollka creates mobility bases in an innovative and modern design. It is possible to match different seating systems and individual seating shells with the mobility bases. At the development we look particularly to simple and easy handling.

The mobility bases ensure mobility at any time. It is easy to move, maneuverable, tip-resistant and partly height adjustable. The patient can sit in a comfortable position at eye level with the environment.

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The next generation of mobility bases

The TOURER – the new generation of mobility bases for seating shells and orthopedic seating systems. The rigid construction looks very light thanks to its modern design. Furthermore, parents and carer can participate active in life with the TOURER System.

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*drumbrake wheels removable
by quick-release axle*

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*seat and back angle are adjustable stepless
by a gas spring*

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- *rigid aluminium frame with abduction*
- *seat frame height adjustable and angle adjustable by gas spring*
- *adjustable back, foldable*
- *seat height adjustment*

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Details

The TOURER is available in rear wheel sizes of 12" / 16" / 20" / 22" / 24".

Furthermore, the seat height is adjustable in a range of 8 cm. The back can be adjusted stepless; this offers a seat depth increase of 7 cm.

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For explanations see last flip page of this catalog.

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The next generation of mobility bases

F1 – the proven generation of mobility bases for seating shells and orthopedic seating systems. The functional and proven construction offers unlimited options in the supply of seating shells.

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height and angle adjustable push handle with self-leveling back adaption

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seat and back angle stepless adjustable by a gas spring

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- *trapezium adapter for seating shell*
- *different footrest variations*
- *back guide*
- *seat extension*
- *wheel cover for rear wheels*

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Details

A huge range of options can be realized individually. Of Course the F1 is tested and licensed for the transport in the BTW.

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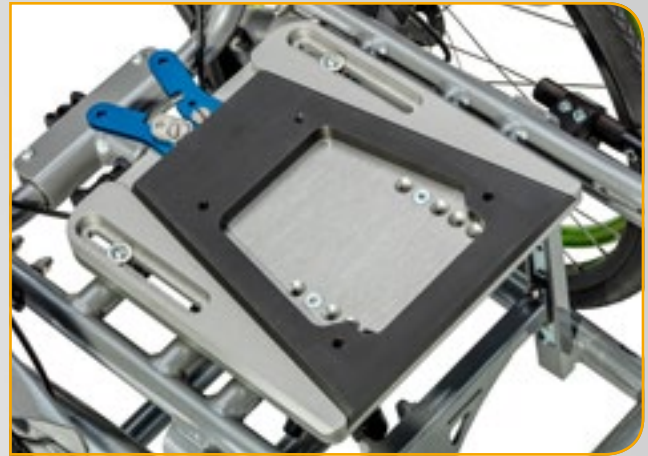
The little seating shell mobility base

With its short frame, the RACER is very agile, and primary used for little shells. Combined with corresponding seating shells, the RACER provides varied individual adjustment possibilities.

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*ergonomic trapezium push bow,
angle and height adjustable*

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- *rigid aluminium frame with abduction*
- *seating frame angle adjustable by gas spring and height adjustable*
- *trapezium adapter as an option for easy adaption of seating shells*
- *back adjustable by gas spring*

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Details

The innovative design combined with an ultra light construction that makes many challenges in the everyday life easier.

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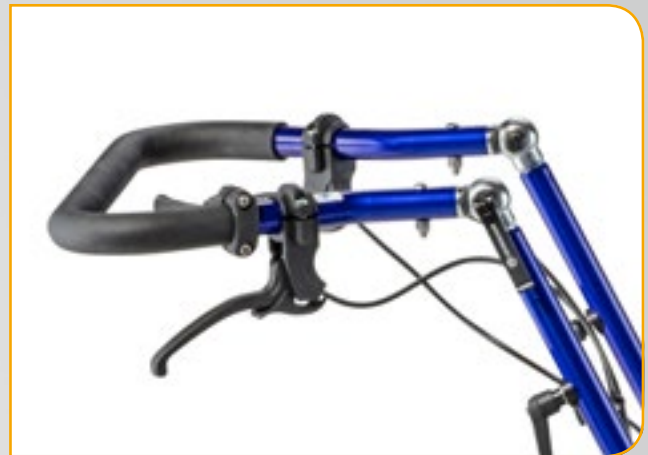
A modern design and clever adjustment possibilities

The innovative CRUISER-module-system is suitable for indoor and outdoor and provides highest mobility. The diverse adjustment facilities and combination possibilities guarantee individual adjustments with relevant seating shells.

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footrest removable with double foot plate, aluminium flat profile

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*seat angle stepless adjustable by a gas spring
back angle adjustable by a gas spring or manuel adjustment*

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- *rigid frame with abduction*
- *seat tilt with gas spring*
- *seat depth adjustable in a range of 7 cm, stepless*
- *back angle adjustable*
- *foldable back*
- *different footrest versions*

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Details

The CRUISER is available with 12" and 16" rear wheels. The back is adjustable stepless, which enables an increase of 7 cm in seat depth.

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A functional design and proven adjustment possibilities

The B2 is the newly optimized basic model and the most affordable product of our mobility bases. The innovative B2 system is suitable for indoor and outdoor and provides highest mobility.

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single footrest with heel flange and knee angle adjustment

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back with handbow, stepless angle adjustable and tiltable.

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- *rigid aluminium frame with abduction*
- *pushbar is foldable and angle / height adjustable*
- *trapezium adapter for seating shell*
- *different footrest versions*
- *back guide*

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Details

The diverse adjustment facilities and combination possibilities guarantee individual adjustments with relevant seat shells.

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Easy driving and functional design

The seating shell mobility base TRAXX TT enables children an active mobility and the greatest possible freedom of movement. The modern and intelligent design combined with lightweight construction makes many things in everyday life easier. The adaption of a seating shell enables children to join the active life with their parents or nurse.

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*trapezium push bow height and
angle adjustable and tiltable*

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*tiltable seating unit and
back angle adjustment*

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- *rigid aluminium frame with abduction*
- *seating frame angle adjustable by gas spring*
- *back angle adjustable and foldable*

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Details

*The TRAXX TT offers a tiltable seating unit and an angle adjustable back unit.
Many accessories can be added for individual requirements.*

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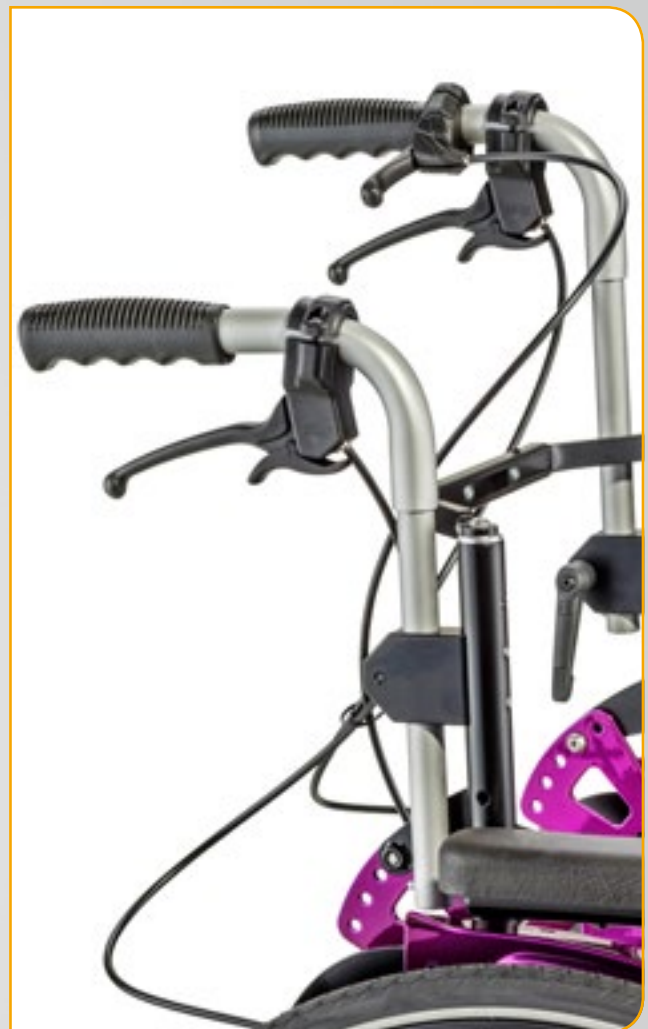
The revolutionary folding mobility base

The revolutionary foldable mobility base ROOKIE enables an active mobility. With the remove of the seat plate, the frame is foldable to a compact size, for an easy transport in the car.

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The ROOKIE can be easily folded by a crossbrace mechanic.

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seat angle adjustable by gas spring and back angle by manual adjustment

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- *foldable aluminium frame with abduction*
- *seat frame angle adjustable by gas spring*
- *back angle adjustable and foldable*
- *trapezium adapter for seating shell*
- *different footrest versions*

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Details

The ROOKIE is available in different rear wheels with dimensions in 20"/22"/24". Optionally, the patient can choose between with or without a drum brake. The ROOKIE seat is easily angle adjustable by a gas spring.

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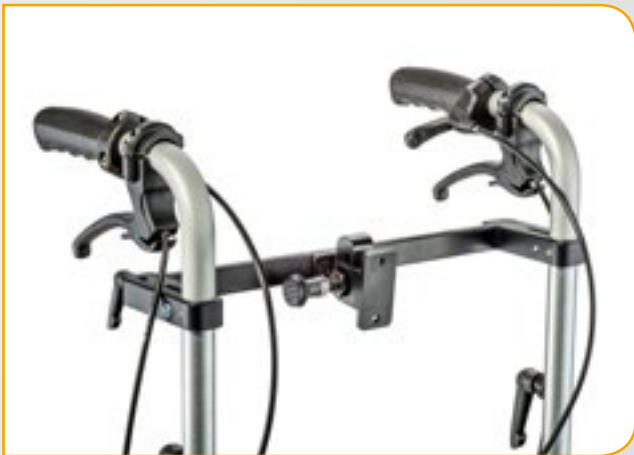
Stability and easy handling for indoor and outdoor

Thanks to the ergonomic frame, the FLOH mobility base obtains a high level of stability and a high degree of flexibility. Due to the stepless height adjustment by gas spring, the seat unit can be lowered from the table height down to the ground. The adjustment of the seat angle is also done by a gas spring.

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*seat height stepless
adjustable by gas spring*

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- *aluminium frame (LT) / steel frame (HB)*
- *seat height and seat angle adjustable by gas spring*
- *ergonomically optimized design*
- *trapezium adapter as an option for adaptation of seating shells*

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Details

The seat tilt is adjusted by a gas spring. The push handle is height and angle adjustable and can be folded forward for loading.

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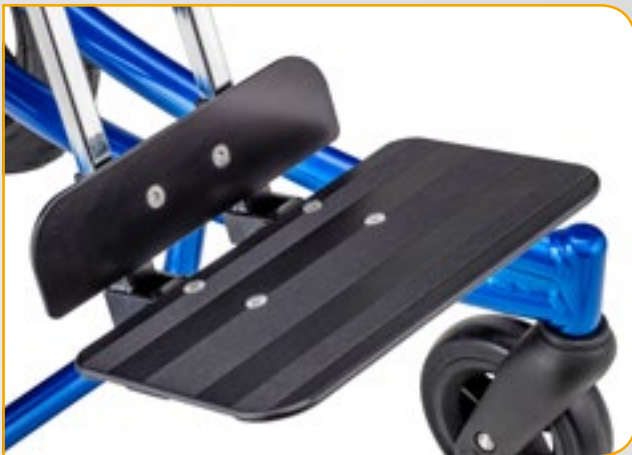
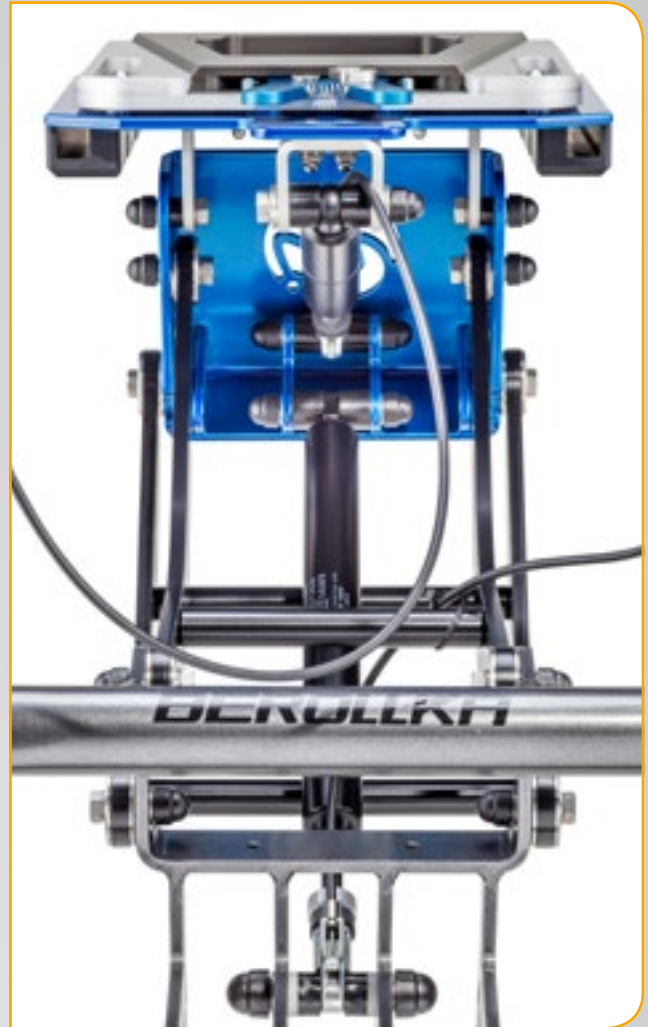
The indoor mobility base for seating shells

The indoor mobility base FLIP allows a stepless height adjustment of the seat system by a gas spring from the table height down to the ground.

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- *aluminium frame (FLIP LT)
steel frame (FLIP HB)*
- *seat height adjustable by gas spring*
- *seat angle adjustable by gas spring*
- *trapezium adapter as an option for adaptation of seating shells*

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Details

Thanks to the ergonomic frame, the mobility base obtains a high level of stability and a high degree of flexibility. This makes it even easier for children to enjoy everyday life.

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Lenkrollen arretierbar
transport wheels, lockable

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Easy handling for supervised locomotion indoor

The indoor mobility base X61 is the ideal solution for the home care sector. This newly designed indoor mobility base is versatile adjustable, safe and extremely manoeuvrable. The low weight guarantees easy handling and the ergonomic X-shape of the aluminium frame provides great legroom and increases the torsional rigidity of frame. The seat frame can be adjusted for different heights.

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Optionaler Trapezadapter
trapezium adapter as an option

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With the daily upright standing, standers can improve the tone regulation, which has a positive impact in the intestinal and bladder function. The standers from Berollka enable various therapy possibilities. They are build by a modular principle. This enable various an optimal and

individual adjustment. The patient gets enough support. In addition, the standers are very flexible to use with the transport wheels.

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Numerous adjustment possibilities

The CAMPUS NANO 630 stander was specially developed for infants and offers all the possibilities of a child-friendly standing therapy. The multifunctional therapy device can be optimally customized to the body measurements of the little ones. Due to the elegant design and the high functionality it meets the requirements for daily use.

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*pivot pelvis frame with pelvis pads, adjustable in
height and depth to be opened from the side*

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*table frame with attachment for
back-pad and headrest*

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- *base frame in U-form with foot plate*
- *single pillar guide to attach the knee pads*
- *table frame with chest, back and side pads*
- *table plate*

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Details

The CAMPUS NANO 630 ALLROUND has an angle adjustment function. With this, the stander can place the patient with an angle of 25° forwards.

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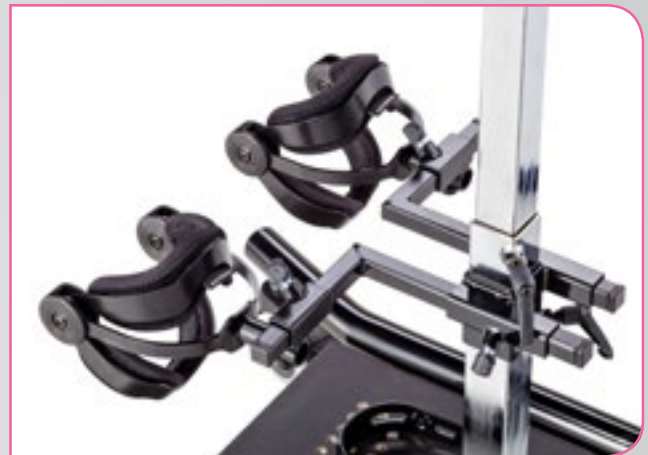
Ready for challenges

The CAMPUS MICRO 628 is developed especially for children and infants. It guarantees optimum, individual adaptation possibilities and easy handling. Thanks to its elegant design and high functionality, it meets the needs of everyday use. Combined with a huge range of accessories and innovative technology, the CAMPUS MICRO 628 becomes the ideal companion during the therapy.

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CAMPUS MICRO in HighLine design with wooden plate multiplex, linoleum surface and pads with contrast flanks

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pivot pelvis frame with pelvis pads, can be opened laterally, adjustable in height and depth

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- *base frame in U-form with foot plate*
- *double pillar guide to attach the knee and pelvis pads*
- *pivot pelvis frame with pelvis pads*
- *table plate*
- *heel shells*

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Details

The CAMPUS MICRO 628 ALLROUND differs from the STANDARD with its angle adjustment function. The stander can place the patient with an angle of 25° forwards.

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Offers ultimate security

The stander CAMPUS 625 guarantees the maximum of security for standing therapy. The extended and multifunctional stander can support or correct body deformations. In addition, the stander can change the psychological perception and attitude. With its shapely design and the high functionality, the stander meets the requirements of the daily use for young people.

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chest pad with integrated side pads, mounted at the table frame, adjustable in height, depth and width.

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- *base frame in U-form with foot plate*
- *double pillar guide to attach the knee and pelvis pads*
- *table frame with chest, back and side pads*
- *table plate*

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Details

The CAMPUS 625 ALLROUND differs from the STANDARD with its angle adjustment function. The stander can place the patient with an angle of 25° forwards.

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Boundless therapy options in the ventral position

The angle adjustable children's stander CAMPUS EASY SWING 671/672/673 offers a huge range of adjustment options and accessories. As a result, with the ventral position stander many individually tailored forms of therapy can be tested.

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* with various centre pillars
** depending on the size

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*beech plywood table with table boarder, multiple
adjustable for therapy options and removable*

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*flexible adaptation by numerous
adjustment possibilities*

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- *base frame with 75 mm transport wheels*
- *chest pad*
- *single or double foot plate and leg pads*
- *angle adjustable by gas spring*
- *pelvis pads, back pads and side pads as an option*

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Details

The centre column can be adjusted stepless from the vertical to the horizontal position by means of a gas spring. Thanks to the optimal position of the centre of gravity, an adjustment of the lying position is easily possible.

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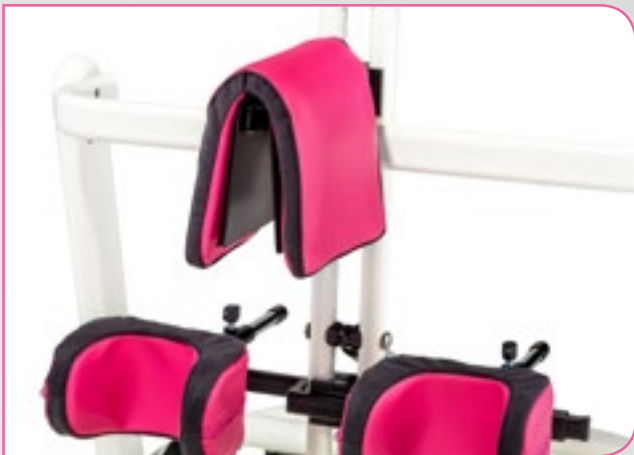
Various adjustment options

The diagonally adjustable back support CAMPUS PHYSIO RELAX offers a huge range of adjustment options and a huge range of accessories for individual forms of therapy.

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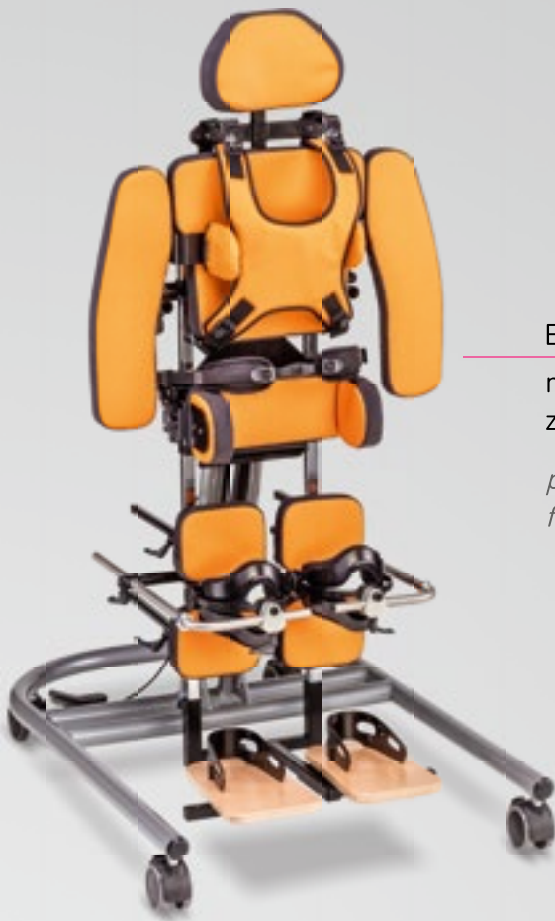
*pelvis retraction strap with ratchet
function for quick adjustment*

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*beech plywood table with
table boarder*

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- *base frame with 75 mm transport wheels*
- *single or double foot plate and leg pads*
- *pelvis and back pad, pelvis side pads, head pad*
- *angle adjustable by gas spring*

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Details

The CAMPUS PHYSIO RELAX convinces with its easy handling. The robust and modern construction offers children and young people a safe therapy form. The power assisting gas spring enables a comfortable angle adjustment of 90° (vertical) till 0° (horizontal).

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To treat in a versatile and safe way

The PHYSIO RELAX MOTION stander now makes it possible for the first time to use the Galileo movement system from Novotec Medical especially for people who need a lot of support and positioning aids.

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For explanations see first flip
page of this catalog.

GALILEO BEWEGUNGSSYSTEM

der Firma Novotec Medical

*Galileo movement system
from Novotec Medical*

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- *base frame in U-form*
- *double pillar guide for adaptation of leg pads, back pad and head pad*
- *angle adjustment by means of adjusting drive*
- *dynamically sprung leg pads and knee pads*
- *adapter frame for the Galileo system*

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Details

Based on the PHYSIO RELAX with countless positioning possibilities, the PHYSIO RELAX MOTION is in combination with the Galileo system a trend-setting vibration training for active standing therapy, according to our guiding principle "Berollka bewegt".

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*For explanations
see last flip page
of this catalog.*

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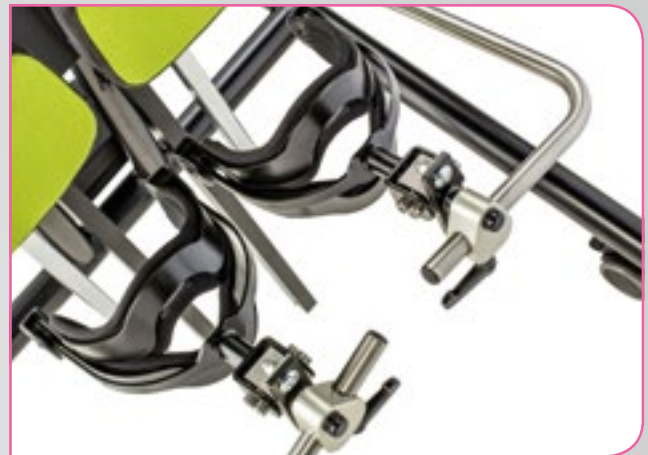
Various adjustment options

The back-lying standing-system CAMPUS RELAX 681/682/683 gives children completely new experiences. Individually adjustable leg pads and foot plates allow a spread of legs.

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*chest strap on four sides easy
to open with a push button*

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- *base frame with 75 mm transport wheels*
- *foot shells or double foot plates and single leg pads*
- *pelvis pad and back pad, pelvis side pad, head pad*
- *angle adjustable by gas spring*

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Details

A gas spring allows stepless adjustment from the standing position to the horizontal supine position. The versatile adjustment options and a huge range of accessories create optimal conditions for individual forms of therapy.

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
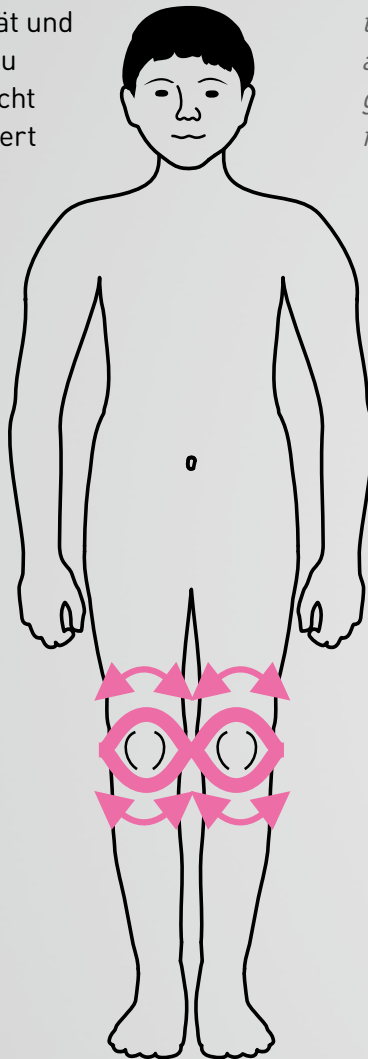
Dynamical mounted positioning

Positioning aids equipped with MOFLEX elements, such as leg pads or knee pads, allows to positioning the person in the stander in a optimal way.

In addition, the dynamically mounted pads support active dynamic alignment and controlled rotational freedom of movement.

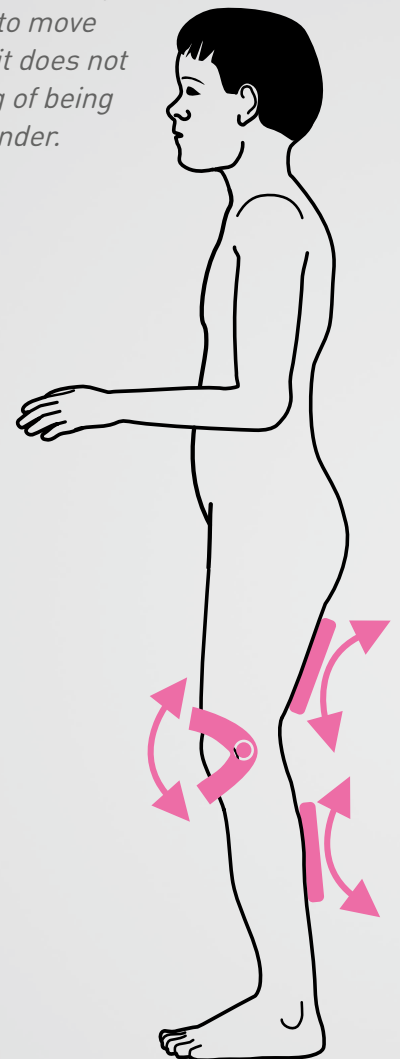In spite of their positioning, they give the patient the flexibility and the possibility to move a little bit and it does not give the feeling of being fixed in the stander.

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By mounting the MOFLEX unit, you will get knee and leg pads that are spherically seated, that means that the pads are flexible and can move up and down, as well to the sides by that they provides a dynamically positioning of the Patient.

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Bully

122

With the right therapy, infants can specifically promoted in their early childhood development. If possible, crawling and straightening should be experienced and worked out for themselves. The crawling and walking aids of Berollka are individually and ergonomically adjustable and

expand the variety of movement and perception playfully. The child's personality can evolve and the way for a preferably self-determined life is paved.

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Flexible support

During the development of the KÖLNER walking stick, care was taken that a successful course of therapy could be adapted to this medical aid. Since the foot of the crutch is dynamically movable, it is possible to put the crutches at an angle, this gives the movement more flexibility. In addition, by reducing the support surface, the activation level is continuously increased. This continual adaptation creates new challenges for the patient.

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werkzeuglose Höheneinstellung mittels Sperrclip
height adjustment without tools by means of a locking clip

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The playful crawling aid for infants

The KRABBLER enables a playful crawling training and increases the freedom of movement and the ability of perception. The crawling movement results in optimal stretching of the pelvis. The chest pad is anatomically shaped and adapts to the upper body of the child. The position of the pelvic element causes the pelvis to tip forward and thus stretch the spine. The centre of gravity in the shoulder girdle, which is an important stabilizing factor for the hand support function, is regulated by the KRABBLER. In addition, the body erection and the process of locomotion of the child are promoted and improved.

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The little racer for little explorers

BULLY is the new mobility base for walking and driving, especially designed for infants. It enables a playful walking training and increases the freedom of movement and the ability of perception. It is characterized by its ergonomic frame and the height adjustable chest support, which, when opened, facilitates entry. The seat height and the height of handlebar can be adjusted individually.

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
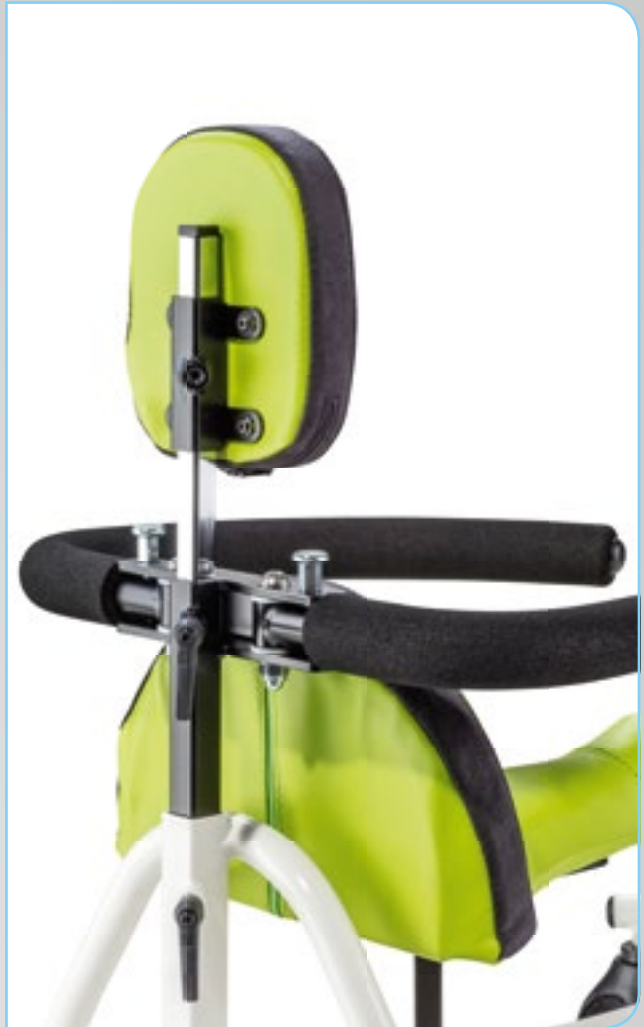
*For explanations
see first flip page
of this catalog.*

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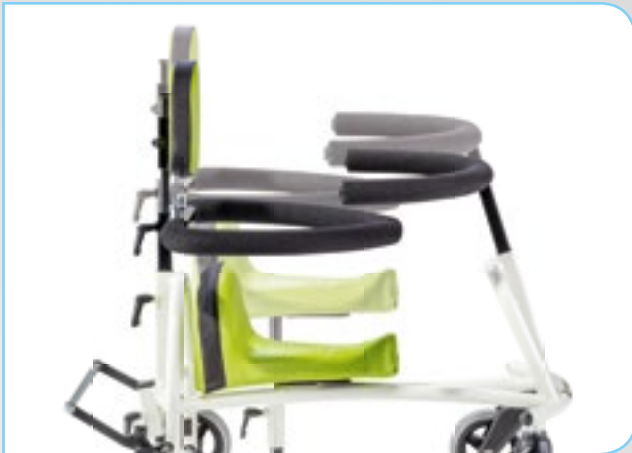Individual requirements involve individual solutions. Varied accessories can support the requirements of patients with wheelchairs, mobility bases or standers; for example positioning aids for the optimal seating position or an outdoor frame for the wheelchair.

With our huge range of accessories, our products can be equipped individually, comfortably and ergonomically.

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Overcome obstacles – break barriers

The outdoor front frame Mono-TRIKE offers more freedom. Overcoming rough terrain and little obstacles present no longer any problems.

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The Mono-TRIKE can be added and removed easily with a quick-release fastener and can be adapted to the wheelchairs KAYOU, FINDUS and FILOU.

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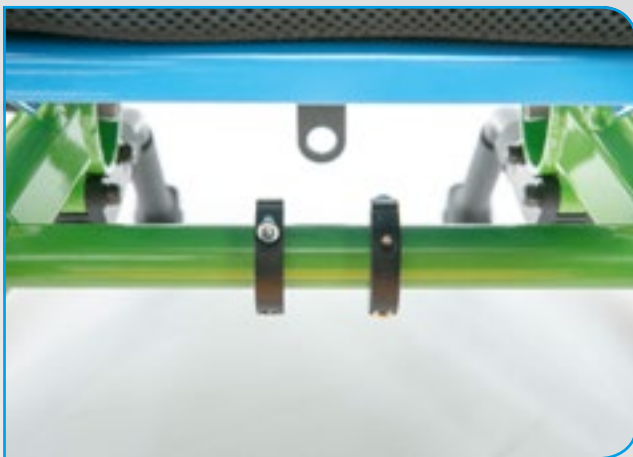
overcome obstacles – break barriers

The outdoor front frame TRIKE offers more freedom. Overcoming rough terrain and little obstacles present no longer any problems.

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The TRIKE can be added and removed easily with a quick-release fastener. In addition, it is compatible with nearly every wheelchair.

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Anatomical sitting

The very lightweight ergonomically shaped back shell is available in two different contour depths of 30 and 50 mm. It is composed of a concave aluminium shell, a cushion and a cover made of breathable fabric. Due to the semi-circular shape of the back plate, the back can be optimally mounted between the back tubes of the wheelchair.

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FIDIBUS back completely covered with breathable fabric. Cover removable

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FIDIBUS back ASR out of aluminium back shell, powder-coated, with upholstery made of foam and breathable fabric, fastened by velcro

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Sitting with pressure relief

The anatomical shape in the seating area through the pelvic cavity and the leg support allows you to sit without tiring. The seat cushion is simply attached on the seat upholstery or the seat plate by means of velcro-fleece connection.

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FIDIBUS seat cushion is a very lightweight ergonomic seat cushion with an integrated aluminium plate and a cover made of a breathable fabric.

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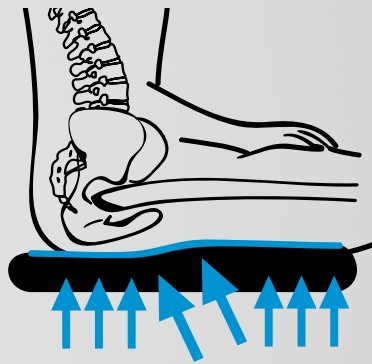
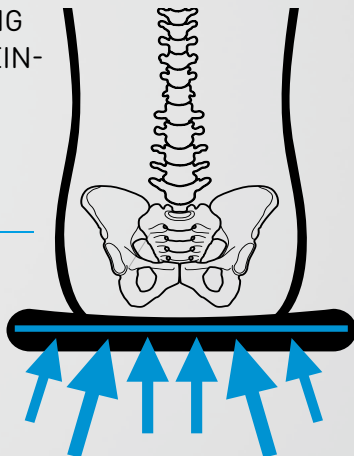
The anatomic seat and back system

The innovative seat and back system MATRIX offers an individual optimal positioning and support for demanding users. The upper body is safely guided with the back system. Postural deformity can be avoided or corrected. The back shells with the breathable cover support the sitting position and allowing the user to sit for long periods of time without complaints.

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little support with 7 cm conture depth

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For explanations see flasflip page of this catalog.

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*medium support
with 12 cm conture depth*

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*great support
with 17 cm conture depth*

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Seat cushion SIMPLIFY

The ergonomic seat cushion SIMPLIFY provides optimum pressure distribution. Through the anatomical shaping, adequate pelvis stability is ensured.

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individual adaptation through modification of the foam contours

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Farbvielfalt
variety of colours

Endmontage
final assembly

Customization is about the implementation of individual medical and therapeutic requirements, with the aim to improve peoples quality of life. In order to meet these individual requirements, we rely on a consistent and permanent exchange of ideas with our customers and partners.

Our team of experienced technicians with the highest level of technical knowledge and engagement translates your requirements into tailor-made solutions.

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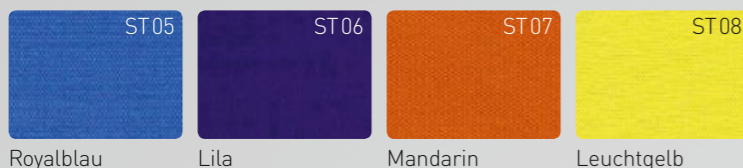
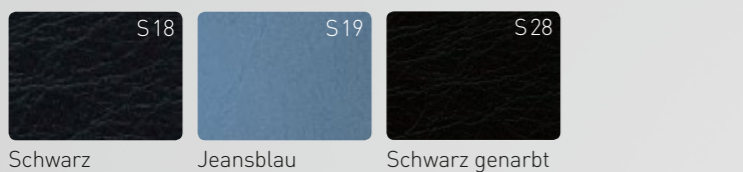
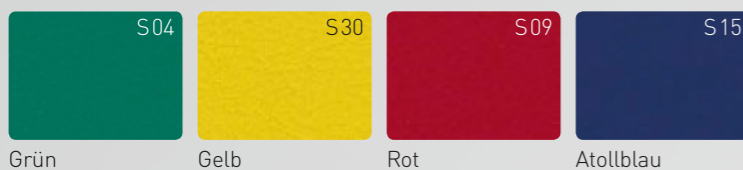
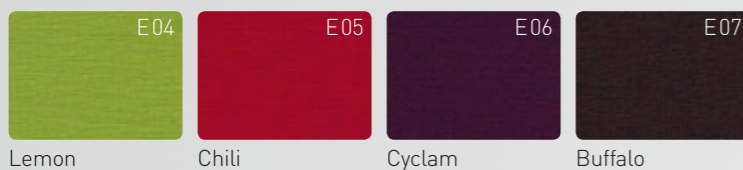
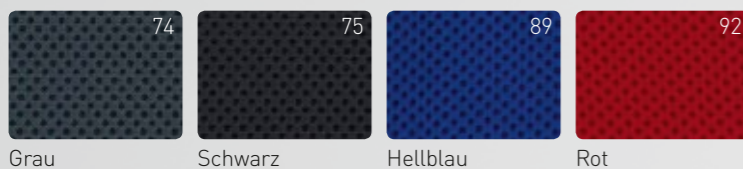
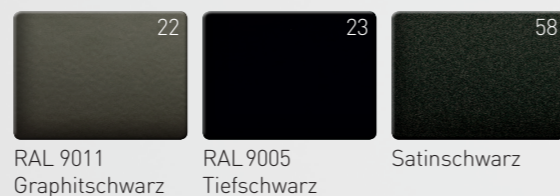
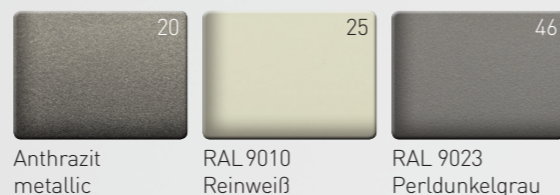
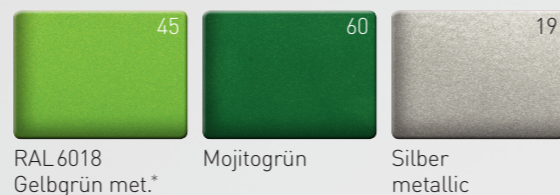
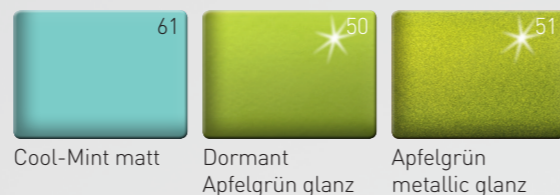
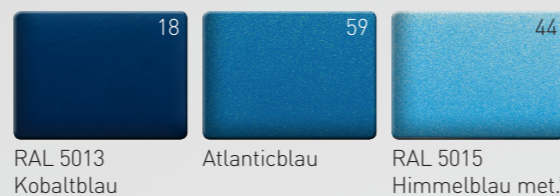
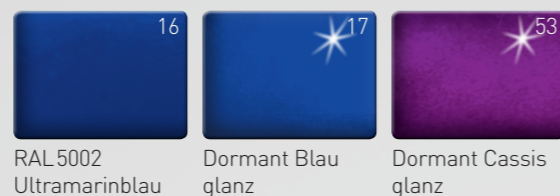
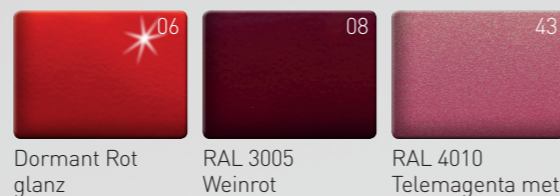
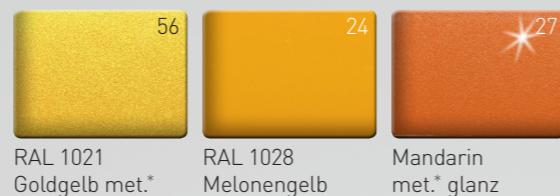
Our surface coating is made of excellent powder coatings. Those powder coatings are characterized by a brilliant optic, consistent high quality and high mechanical—as well as UV-resistance to ensure for you to be able to enjoy your product for a long time.

Should you be interested in a colour from our spectrum which is—according to the order form—not offered for your desired product, you can of course order it as a colour option at a surcharge.

We have made every effort to show all colours as realistic as possible. However, the visual colour impression can vary from reality due to different media (e.g. monitor display, print etc.).

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